




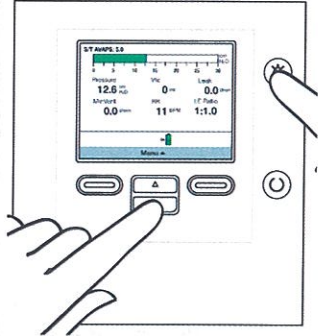

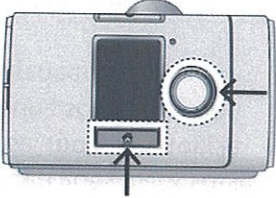

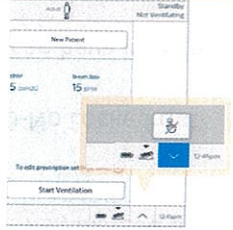


Home Ventilator Comparison Chart

VENTILATOR	A40	STELLAR	AIR CURVE	Breas Vivo 45 LS	EV 300/EVO
INVASIVE/ NONINVASIVE	Noninvasive	Noninvasive	Noninvasive	Invasive	Invasive
BATTERY	YES Detachable and/or battery pack	YES Internal only	MAYBE Will only have battery if dependent on ventilation. Otherwise, no battery.	YES Internal, detachable and or battery pack	YES Internal and detachable
BATTERY LIFE	Up to 3 hours	Up to 2 hours	Up to 8 hours (humidifier and heated tubing off)	Up to 6 hours	Up to 7.5 hours
QR CODE FOR VIDEO LINK			 AirSense and AirCurve are very similar.		 Video of EVO but it has the same screen as EV300
LOCKED SCREEN/ SECRET HANDSHAKE	 Press and hold until beep to unlock. It will lock automatically once out of menus.	 Press and hold until it beeps and then select 5 mins to unlock. Press and hold to lock after.	 Press and hold until it shows settings option to unlock Select "exit clinical menu" to lock.	 Press and hold the right and left arrows. You can switch between clinical mode and home.	 Touch the screen then press yes for 3 seconds to unlock. Locks automatically after 5 mins. EVO -Push "Alarm Silence & Clock icon (bottom R)"

Home Ventilator Comparison Chart

VENTILATOR	A40	STELLAR	AIR CURVE	BREAS VIVO 45 LS	EV 300
KEY POINTS TO REMEMBER	<ul style="list-style-type: none"> -Can be used without humidifier during travel. 	<ul style="list-style-type: none"> -Remember the extra step to unlock (select 5mins) -Scroll through pages to find info -Can be used without humidifier during travel <p><i>*Leak compensated*</i></p>	<ul style="list-style-type: none"> -Use 2 pot technique to refill or machine will turn off due to high leak. -Screen goes black while in use, move the toggle to bring screen back on when needed. -Humidifier has to be emptied and humidifier turned off to travel with this machine. <p><i>*Leak compensated*</i></p>	<ul style="list-style-type: none"> - Should do Pre-test only when changing circuit - Circuit disconnect check must be done with no circuit attached. Disconnect at expiratory filter. - Obstruction alarm must be done without whisper swivel in line <p><i>*Leak compensated*</i></p>	<ul style="list-style-type: none"> -In PC-SIMV mode you will now set a PC which will be added to PEEP to get the PIP, instead of setting IPAP/PIP -EV300-No Locked out mode. -EVO- can have lock set -Can add Prescription accidentally on main screen. Move to garbage can to delete. <p><i>*Leak compensated*</i></p>
HEATED TUBING	Yes	No	Yes	Yes	Yes
Alarms	Circuit Disconnect Low Circuit Leak	Circuit Disconnect (built-in) High Leak Tube Blocked Non -Vent Mask	High Leak Non-Vent Mask	Circuit Disconnect Obstruction/rebreathing Low pressure	Circuit Disconnect Obstruction